
# EZee Life

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# Item  
CRP0501

# Ref  
CH1040

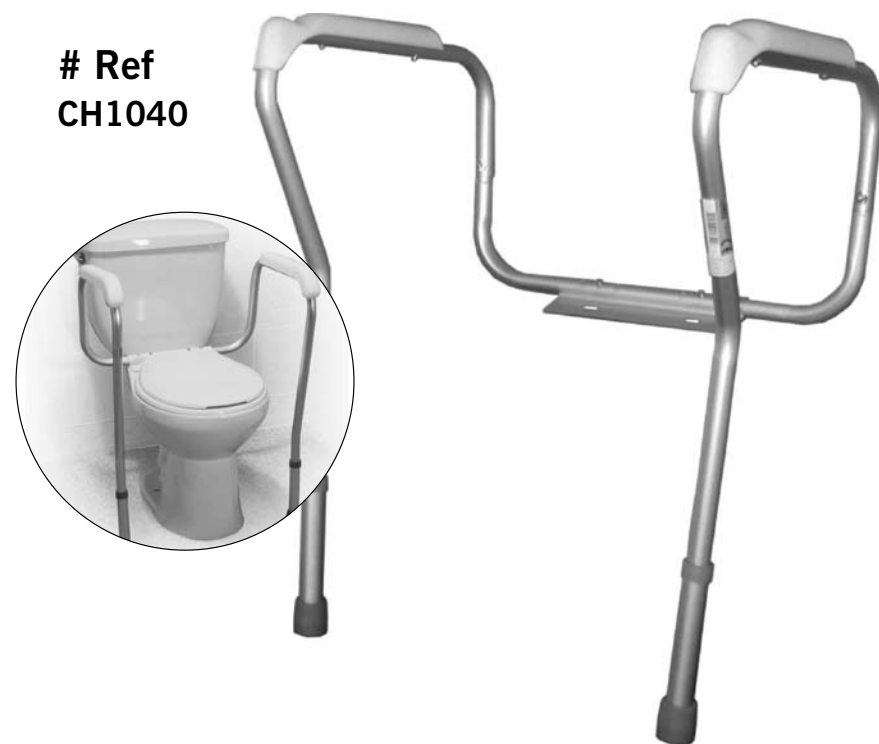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MFG #  
CH1040

### Limited Warranty

**EZee Life** brand product have a Limited Warranty for one year from the date of purchase against faulty workmanship or materials.

In the event of a defect covered by this limited warranty, we will, at our option, repair or replace the device. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or negligence or non-observance of the instructions set out above.

If you have a question about your **EZee Life** device or this warranty, please contact an authorized **EZee Life** dealer.

# EZee Life

## Features

- Anodized aluminum is sturdy and lightweight
- Waterfall armrest provide additional comfort and support
- Arm height and width are adjustable
- Height adjusts from: 28" - 32"
- Width adjusts from: 18"- 25"
- Weight capacity: 300 LBS

## Assembly Instructions

1. Detach your toilet seat by first removing toilet seat bolts.
2. Place the rail mounting flange on the toilet and align the holes (Figure 1).
3. Place your toilet seat on top and align holes (Figure 1).
4. Realign bolts and tighten securely.
5. Slide assist rails into Flange Bride and secure them with built-in snap buttons. (Figure 2)
6. Set rail to desired width and height using the built in snap buttons. Make sure all built in snap buttons are secured before use of product (Figure 4).

Figure 1

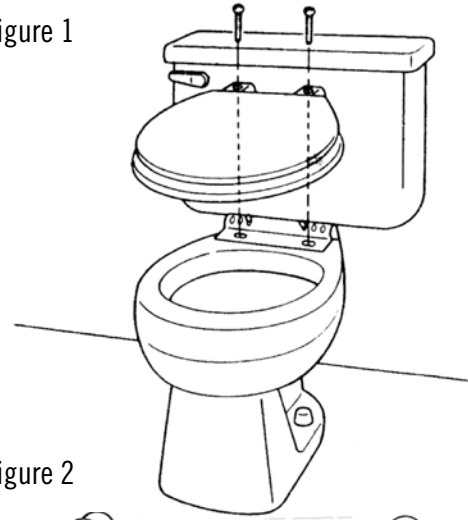
Figure 2

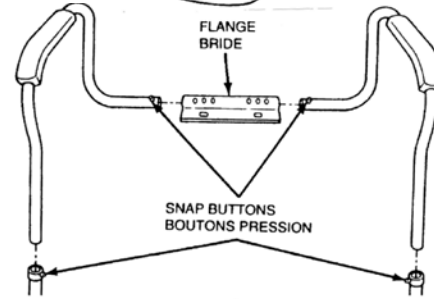
Figure 3

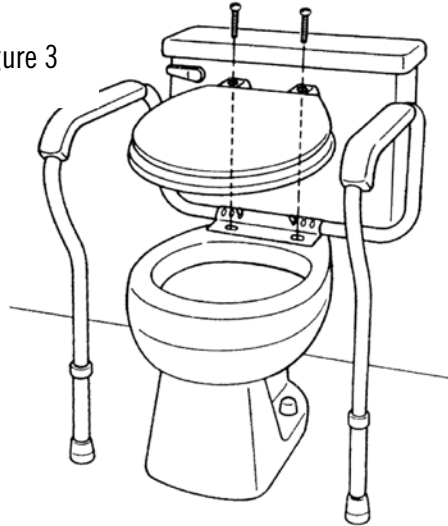
Figure 4

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