# **EZee Life**

# **Wide Seat Drop Arm Commode**

Model: CH1052



# **Limited Warranty**

Ezee Life™ products carry a one year warranty against manufacturing defects, faulty materials and workmanship.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the item. Please contact your supplier/dealer for assistance should a defect become apparent.

The warranty does not extend to consequential costs resulting from the fault or defect in the product, in particular freight and travel costs, loss of earnings, expenses or other expenses which may occur.

The warranty does not cover normal wear and tear resulting from normal use or damage caused by accident, misuse, negligence or improper installation or assembly.

The vendor/manufacturer will not accept responsibility for any damages or injury caused by misuse, negligence or non-observance of the instructions supplied.

If you have a question regarding your product, its use or this warranty please contact an authorized dealer.

220 Wildcat Rd., Toronto, ON M3J 2N5



CH1025 Wodel:

"9fid 99S3

# Instructions



## **Features & Specifications:**

Drop arm design for easy transfers

Bucket and lid included

Splash guard for over toilet use included

Powder coated steel frame

Colour: Gray

Width: 25.5" (65 cm)

Adjustable Seat Height: 19" to 23" (48.3 to 58.5 cm)

Weight Capacity: 400 lbs. (181.8 kgs.)

#### Care and Cleaning:

To maintain the beauty of the original finish, clean with mild soap and rinse with warm water.

Wipe commode dry after use if required.

### **Precautions:**

Periodically check assembly for continued tightness.

NEVER USE BATH SEAT IF ANY LOOSENESS IN THE ASSEMBLY IS NOTICED AS PERSONAL INJURY MAY RESULT.

NEVER STAND ON THE BATH SEAT. A dangerous fall could result. CAUTION: Never exceed maximum weight limitation of 400lbs.

