# EZee Life

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Conforme aux normes ANSI A117 1.

EZee Life, 252 Wildcat Road, Toronto, ON Canada M3J 2N5

# EZee Life

## Bath Bench with Plastic Seat (Tool Free)

Item #      MFG #  
CRP0190      CH1060TF

### Limited Warranty

**EZee Life** brand product have a Limited Warranty for one year from the date of purchase against faulty workmanship or materials.

In the event of a defect covered by this limited warranty, we will, at our option, repair or replace the device. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or negligence or non-observance of the instructions set out above.

If you have a question about your **EZee Life** device or this warranty, please contact an authorized **EZee Life** dealer.

### Cautions

**WARNING:** Use only the stainless steel hardware provided to avoid decolorization and dangerous weakening of the hardware from rust or corrosion.

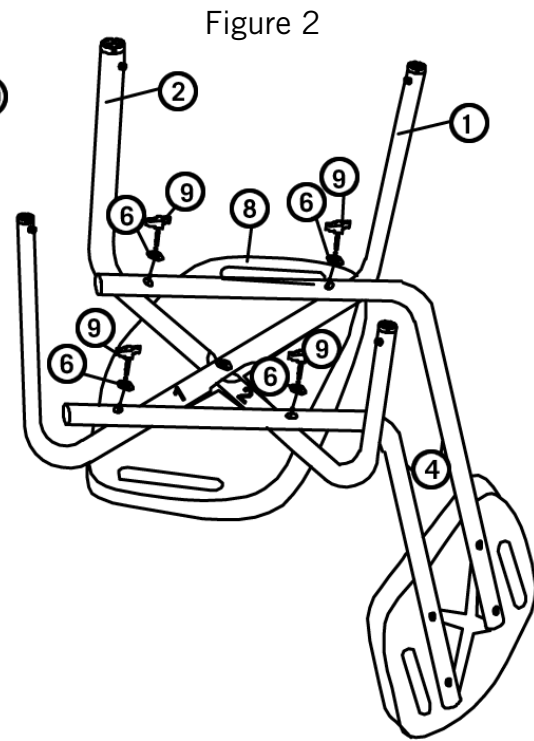
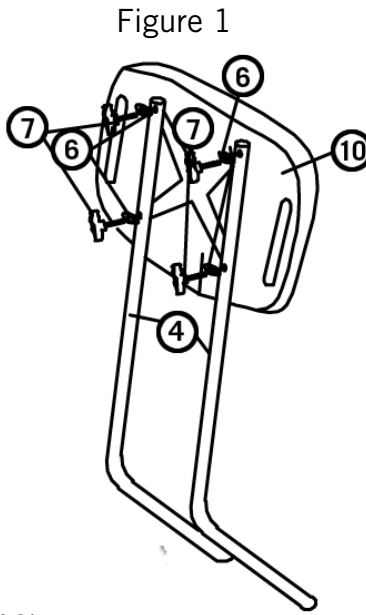
Periodically check assembly for continued tightness. NEVER USE BATH SAFETY SEAT IF ANY LOOSENESS IN THE ASSEMBLY IS NOTICED, AS PERSONAL INJURY MAY RESULT. SIT. DO NOT STAND on this seat to avoid a possible dangerous fall.

Meets ANSI Standards A117 1 Specifications.

EZee Life, 252 Wildcat Road, Toronto, ON Canada M3J 2N5



# EZee Life

Item	Description	QTY
1	U bending tube	1
2	U bending tube	1
3	Adjustable foot tube	4
4	Backrest tube	2
5	Small t-screws	1
6	R-washer	9
7	medium T-screws	4
8	Seat	1
9	Large T-screws	4
10	Back	1

## Assembly Instructions

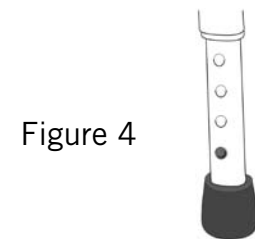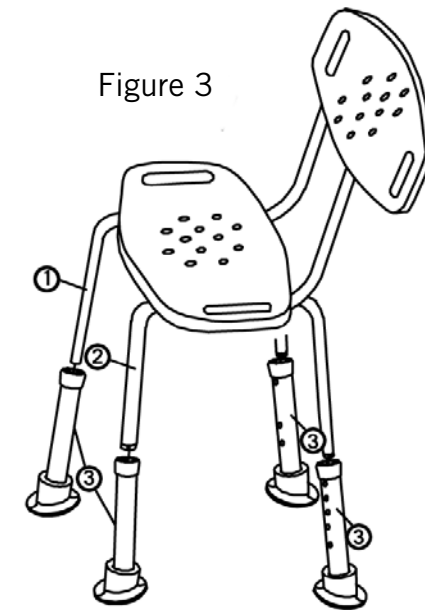1. Place the back rest upside down (item 10).
2. Line up (item 4) on the back of the backrest (item 10) (Figure 1).
3. Insert medium T-screws (item 7) into R-washer (item 6) and then into hole on (item 4) and backrest (item 10) and fasten (Figure 1). Repeat step for remaining holes.
4. Place the seat upside down (item 8).
5. Put U bending tube (item 1 & 2) on the bottom of the seat (item 8) marked with same numbers 1 & 2 (Figure 2). Then line up tubes for backrest (item 4) with holes on (item 1 & 2) (Figure 2).
6. Insert large T-screws (item 9) into R-washer (item 6) and then into hole on (item 4 & 1 or 2) and bottom of seat (item 8) and fasten (Figure 2). Repeat step for remaining holes.
7. Insert small T-screws (item 5) into R-washer (item 6) and then into hole on (item 1 & 2) in the middle of seat (item 8) and fasten (Figure 2).
8. Place item 3 (Adjustable foot tube) onto item 1 or 2 until you hear a click (Figure 3). Repeat for the other foot pieces.
9. Set height using the built in snap buttons. Make sure all built in snap buttons are secured before use of product and at the same height (Figure 4).

## Care and maintenance

- To maintain the beauty of the original finish, clean with mild soap and rinse with warm water.
- Wipe bath bench dry after each use.
- Always check insert screws to make sure they don't loosen over time.

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