

EZee Life™

Bath Chair with Removable Back

Model: CH 1064



Model: CH 1066



EZee Life™ Products

220 Wildcat Road, Toronto, ON M3J 2N5



Specifications:

Weight capacity 300 lbs. (136 kgs)
Slight contour seat with drainage holes
Extra hole for placement of hand held shower head
Non slip feet for extra safety
Durable rust proof aluminum frame
Foot print 16.5" (42cm) x 16.5" (42cm)
Seat width 19.5" (49.5cm) (red handle removed)
CH 1064 Seat depth 13" (33.0cm)
CH 1066 Seat depth 13" (33.0cm)
Seat height 13.5" (34.3cm) -17.5" (44.5cm)
CH 1064 Back height 15" (38.1cm)
CH 1064 Back width 17" (43.2cm)

Features:

Adjustable height seat
Maintenance free aluminum frame
Easy to clean moulded seat and back
Light weight
300 lb (136 kgs) weight capacity
Support handle can be mounted on either side

Limited Warranty:

Ezee Life™ products carry a one year warranty against manufacturing defects, faulty materials and workmanship only.

Any alterations, misuse, abuse, or accidental damage voids this warranty. Ezee Life™ reserves the right to replace or repair any part that has become defective.

The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur.


Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

Operating Instructions

Adjusting the Seat Height

1. Depress the pin lock button (1).
2. Slide the lower leg tube to the desired height and line up the adjustment holes (2) with the pin lock and allow the pin to pop through.
3. Repeat this procedure for all four legs.

Note: Ensure all four legs are set to the same height. Failure to set all the legs the same height will make the bath chair unstable and could lead to an accident.

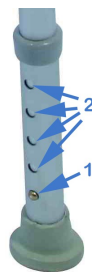

Attaching the CH 1064 Back

The back rest for the CH 1064 simply snaps into place. Insert into backrest slot and push down.

Safety Information

Ensure there are no loose or damaged parts before using this bath chair.

Ensure the rubber leg tips are not loose or damaged before using the bath chair. If loose or worn, replace or repair.

To avoid tipping, DO NOT attempt to reach objects that are at a distance that require you to lean towards what you are reaching for.

To avoid tipping, DO NOT lean over the back of the bath chair.

DO NOT sit or transfer into the bath chair unless it is fully open.

Use caution on slippery surfaces.

DO NOT stand on seat.

DO NOT overload this bath chair. The weight capacity is 300 pounds (136 kgs).