# EZee Life™

## CAUTION!

The overbed tables are not designed to support heavy weight. Users must not lean on them for support when getting up or sitting down. Max Load 10 lbs.

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## Limited Warranty

Ezee Life products carry a one year warranty against manufacturing defects, faulty materials and workmanship only. Any alterations, misuse, abuse, or accidental damage voids this warranty. Factory Direct Medical reserves the right to replace or repair any part that has become defective. The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur. Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

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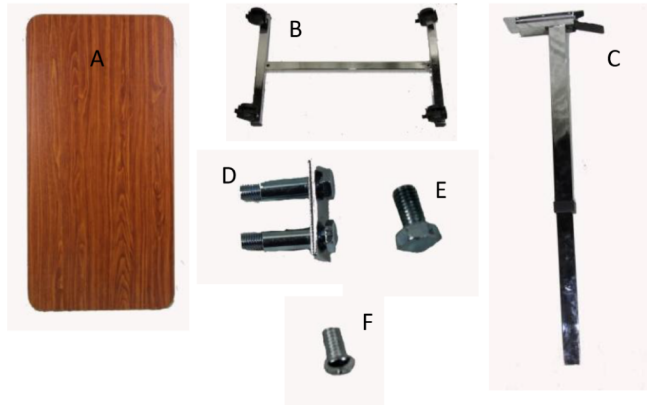
3

## ATTACH BASE TO POST - FIXEZ LA BASE AU POTEAU

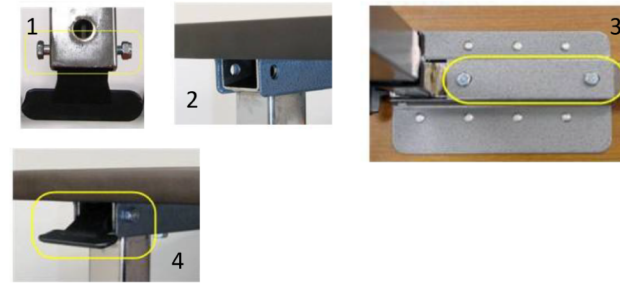
Insert the bolts and plate combination through the base to attach post. Use 17mm wrench to tighten

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
Unscrew existing bolt, nut and ratchet lever from post with 10 mm socket wrench (1)  
 Slide the table top into the post (2)  
 Attach table top to post using two small bolts (3). Tighten with 13mm wrench  
 Reassemble bolt, nut and ratchet lever to post (4)

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5

## REPLACING THE SCREW - REMPLACEMENT DE LA VIS

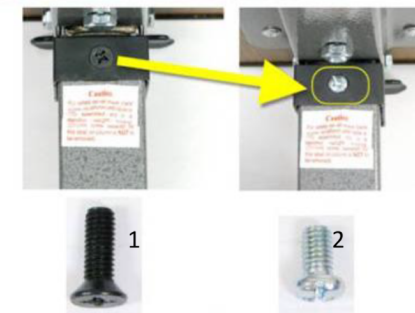Remove the black locking screw (1) of the post and replace it with silver coloured screw (2).

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6

## OPERATING INSTRUCTIONS- INSTRUCTIONS D'UTILISATION

(1) Shows the overbed table fully assembled  
 The table height can be adjusted by squeezing the ratchet lever towards the table top (2)  
 Using your other hand support the other end of the table top while adjusting the height

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