# EZee Life

## Important Information

1. Read the instructions before assembling; do not use this product without proper instruction from a professional.
2. If components are damaged or missing, contact your dealer immediately.
3. Do not use raised toilet seat unless it fits tightly on toilet bowl.
4. Always check for secure fit of raised toilet seat.
5. This product has a maximum user weight of 135kg / 300lbs.

## Intended uses

Toilet aids increase independence when using the toilet, particularly by compensating for impaired movement.

## Installation Instructions

1. Raise the toilet seat that is attached to the toilet. The Raised Toilet Seat should be placed on the rim of the toilet bowl.
2. Place the Toilet Seat upside down on a flat surface and slop one of the L-shaped brackets into one of the slots in the side of the seat (A). Push lightly on the bracket and run the knob clockwise until the thread engages. Repeat with the second bracket.
3. Turn the knobs until the brackets are farther apart than the width of the toilet bowl.

4. Turn the seat over and place it on the rim of the toilet bowl so that the narrow cutout is at the front and the brackets are near the back. Position the seat with the front lip on the seat against the outside front of the bowl.
5. Tighten both brackets until they are snug against the bowl.
6. The Raised Toilet Seat should be level and fit tightly.
7. Regular checks of the brackets should be made, ensuring that the toilet seat is secure. Spare brackets are available if required (CHP2020BRACKETS).

## Care and Cleaning

- All parts can be cleaned with soap and hot water (up to 195°F / 90°C).
- The adjusting screws can be popped out of the brackets for cleaning.

- Cautions :**
- Do not hold onto the seat as a support when getting on and off the seat.
  - Be sure the toilet seat is securely in place before putting weight on it.

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