

# EZee Life™

## Steel Bathtub Safety Rail

Model: CH2039



EZee Life™ Products

34 Futurity Gate, Unit 15  
Concord, Ontario L4K 1S6



## Limited Warranty:

Ezee Life™ products carry a one year warranty against manufacturing defects, faulty materials and workmanship only.

Any alterations, misuse, abuse, or accidental damage voids this warranty. Ezee Life™ reserves the right to replace or repair any part that has become defective.

The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur.

Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

## Installation Instructions

1. Clean Bathtub thoroughly and wipe dry before applying rail. Contact surfaces must be dry and free of soapy residue to assure secure installation.
2. Deploy knob and open until rail is wide enough to slip on side of bathtub. 3.
3. Place Safety Rail on bathtub with knob facing outside of tub and tighten until rail feels tight and secure. 4.
4. To test installation of rail, pull forward, back and side to side. If rail slips or moves, make sure surface is free of soapy film and is dry, then replace at desired location and retighten.

### Notes:

**Safety Rails require retightening from time to time.**

**Check periodically before use and retighten as needed.**

### CAUTION

**Please use extreme care when installing on fiberglass tubs.**

**DO NOT OVERTIGHTEN as the clamp may crack the fiberglass.**

**Imported / Distributed by:**

Factory Direct Medical  
34 Futurity Gate, Unit # 15  
Concord Ontario L4K 1S6  
Local: 416-739-8393  
Toll Free: 855-235-2400