EZee Life[™]

PERIODIC BRAKE ADJUSTMENT - AJUSTEMENT PÉRIODIQUE DES FREINS

Over time the brake cables will stretch and require adjustment. With the brakes NOT applied hold the adjusting barrel (B) in place and rotate the adjusting nut (N) clockwise to increase brake force, rotate counter clockwise. IMPORTANT: before using the walker again, ensure the brakes are working properly. If the wheels turn while the brakes are applied do not use the walker.

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Safety Precautions

- Consult your doctor, therapist, or other qualified professional to determine if this mobility device meets your specific needs.
- Before using, make sure all parts are secure and the frame is fully opened

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Limited Warranty

Ezee Life products carry a one year warranty against manufacturing defects, faulty materials and workmanship only. Any alterations, misuse, abuse, or accidental damage voids this warranty. Factory Direct Medical reserves the right to replace or repair any part that has become defective. The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur. Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

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nstructions

INSTALL AND ADJUST THE HANDLE BARS - INSTALLER ET RÉGLER **LES GUIDONS**

Stand the rollator upright and open the frame by pushing down on the folding mechanisms until they are flat. (1)

Insert the handlebars. Make sure the brake cables are on the outside of the frame (2) For proper height adjustment, stand upright with shoulders relaxed and arms at your side. Handles should be where you would wear a watch. (3)

Secure the handlebars by inserting the hand screw into the hole in the frame. The hand screw should be flush with the frame. Hand tighten only. (4&5)

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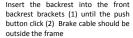
Lock the brakes by pushing down on the brake handles until you feel the brakes click into locked position (1)

To fold remove the basket and pull up on the strap (2)

To unfold push down on the folding mechanism

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Push down on both brake handles to lock the wheels. (1) CAUTION The rear wheels will lock but it may still slide if the floor surface is slippery.

REMEMBER to push down (on the brake handles) before you sit down!

Squeeze both brake handles to release the brake (2) CAUTION never release the brakes until you are standing stable and in the ready walk position. Do not release the brakes if you are sitting down.

REMEMBER if you are walking and wish to slow down, squeeze the brake handles slightly to put brake pressure on the wheels.

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