

ATTO ACCESSORIES are created specifically to enhance, support and work with the ATTO mobility scooter in all user modes; while driving, while wheeling along as trolley and when folded for storage or transportation.

MORE FOR ATTO

movinglife®




ATTO
ESSENTIALS POUCH

Essentially all you need!

A

ATTO ESSENTIALS POUCH IN ATTO CONNECTED MODE

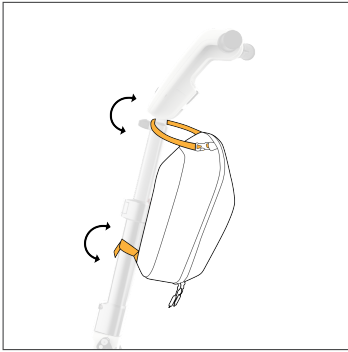

1

- A.** Wrap the top strap round the small gap under the driving console and secure buckle
- B.** Secure the Velcro lower strap around the tiller



2

Once attached you can position ATTO Essentials Pouch in any direction and adjust tiller height freely




3

Position pouch facing outward to stay connected when ATTO is folded


B

ATTO ESSENTIALS POUCH IN SLING CARRYING MODE

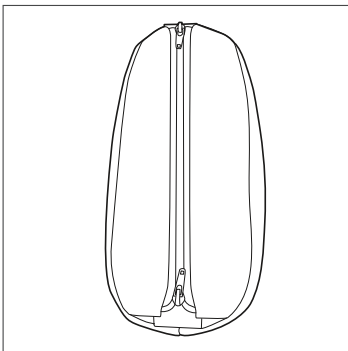


1

Detach ATTO Essentials Pouch

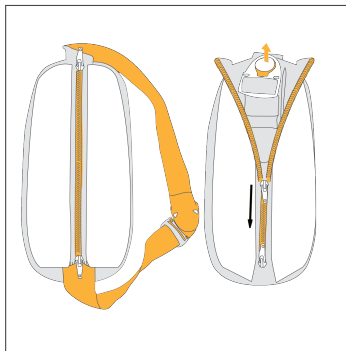

2

Extract Sling strap from behind Zippered Mesh Cover and zip back up (note that you will need to unbuckle and pull out one end from under the mesh cover)

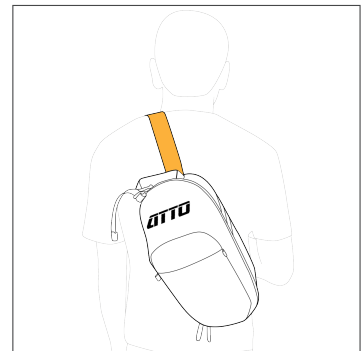

3

Enjoy comfortable sling carrying