

ATTO ACCESSORIES are created specifically to enhance, support and work with the ATTO mobility scooter in all user modes; while driving, while wheeling along as trolley and when folded for storage or transportation.

**ATTO** MORE FOR

movinglife®


**ATTO**  
FLIGHT KIT

Guarantee ATTO a safe journey!




# A

## INSTALLING ATTO FLIGHT KIT FOR ULTIMATE PROTECTION

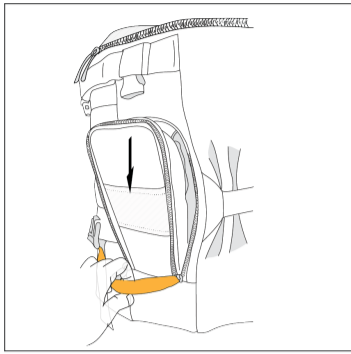


**1**

Pull to open zipper slider to ensure battery space filler is free (don't forget to remove your ATTO battery)

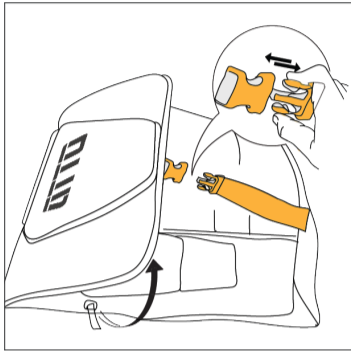


**2**

Unbuckle flap

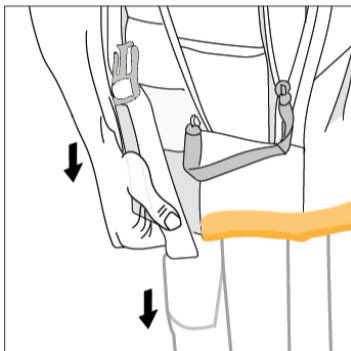


**3**

Unfold Flight Kit and slip over folded ATTO from the top

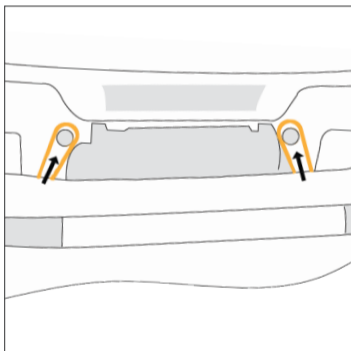


**4**

Connect the two loops to the hooks on the backrest to secure Flight Kit positioning

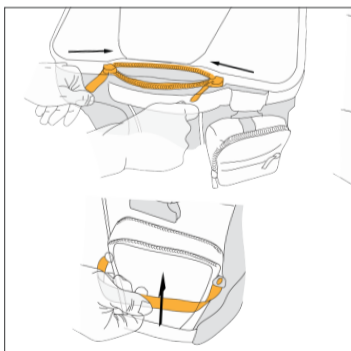


**5**

Zip up the top flap and battery space filler

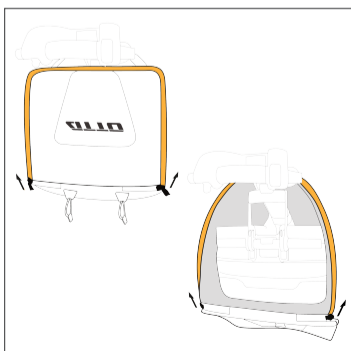


**6**

Or extend ATTO's trolley handle and then close the zippers around it

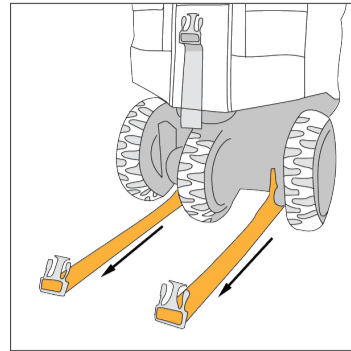


**7**

Convey the two straps under ATTO chassis between the wheels (please be careful while you do so)

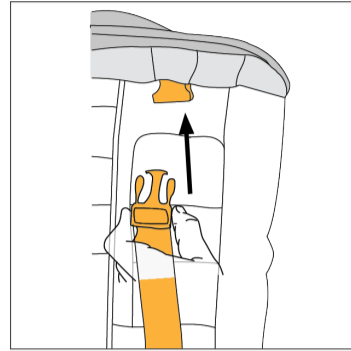

**8**

Thread the straps through the two dedicated loops on the pockets and buckle up

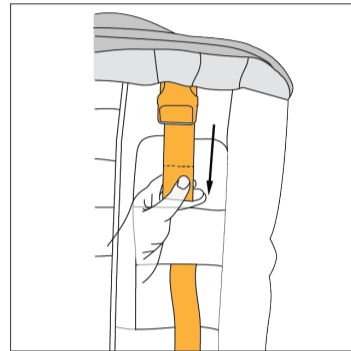


**9**

Pull straps to secure in place


# B

## CARRYING ATTO FLIGHT KIT WHILE IN DRIVING MODE

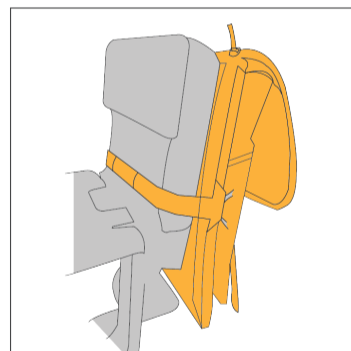


**1**

Fold ATTO Flight Kit, fold flap, buckle all straps out of the way and slip over ATTO backrest. Alternatively store under your seat by slipping the sleeve over it

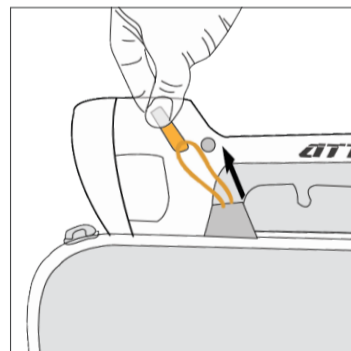

**2**

Connect the two loops to the hooks on the backrest to ensure Flight Kit positioning

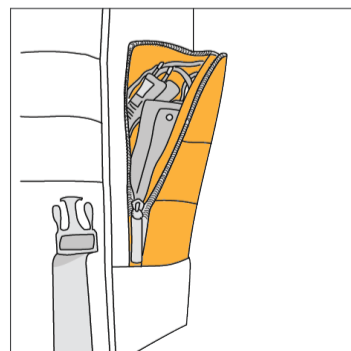

**\***

Note additional pocket available to storage charger