# EZee Life™

---

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**2****TOOLS REQUIRED: None**  
**OUTILS REQUIS : Aucun**

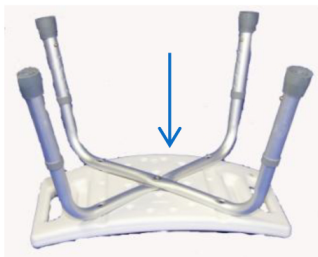
**3** Attach the two sets of legs to the seat. Loosely fasten the middle screw first and then the other four before going back and tightening the middle the rest of the way

**NOTE:** to ensure a proper fit the set of legs on the bottom should have the concave (dent-like) form so that the top set of legs fit nicely in place

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PUSH IN & CLICK  
INSÉREZ ET CLIQUEZ

## Safety Precautions

1. Consult your doctor, therapist, or other qualified professional to determine if this mobility device meets your specific needs.
2. Before using, make sure all parts are secure and the frame is fully opened

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## Limited Warranty

Ezee Life products carry a one year warranty against manufacturing defects, faulty materials and workmanship only. Any alterations, misuse, abuse, or accidental damage voids this warranty. Factory Direct Medical reserves the right to replace or repair any part that has become defective. The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur. Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number

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