

EZee Life™

Economy Rollators

Model: CH3010 – Standard Blue

CH3020 – Standard Red

CH3031 – Low Seat Red



CH3020



CH3010



CH3031

EZee Life™ Products

34 Futurity Gate, Unit 15, Concord, Ontario L4K 1S6



Specifications:

Specification	CH3010	CH3020	CH3031
Frame Material:	Aluminium	Aluminium	Aluminium
Weight:	17.9 lb	17.9 lb	18.3 lb
Wheel Size:	8"	8"	8"
Seat Width:	13 3/4"	13 3/4"	13 3/4"
Seat Depth:	13 1/4"	13 1/4"	13 1/4"
Seat Height:	22 1/4"	22 1/4"	18 1/2"
Width between handles:	17 1/2"	17 1/2"	20 1/2"
Handle Height:	36" to 39 1/4"	36" to 39 1/4"	35 3/4" to 40"
Weight Capacity:	264 lb	264 lb	220 lb
User Height (approx.)	5' 8" to 6' 1"	5' 8" to 6' 1"	4' 6" to 5' 4"

Features:

- Lightweight
- Easily Manoeuvrable
- Padded Flip Up Seat
- Padded Backrest Bar
- Mesh Basket
- Hand Brakes
- Solid Flat-Proof Tires
- Aluminum Frame
- Economically Priced

Limited Warranty:

Ezee Life™ products carry a one year warranty against manufacturing defects, faulty materials and workmanship only.

Any alterations, misuse, abuse, or accidental damage voids this warranty. Ezee Life™ reserves the right to replace or repair any part that has become defective.

The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur.

Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

Rollator Parts Guide

A – Brake Cable

B – Backrest

C – Backrest Bracket

D – Carry Pouch

E – Hand Grip

F – Brake Lever

G – Handle bar hand screw

H – Cane Holder (top pc.)

I – Seat


J – Cane Holder (lower pc.)

K - Wheel



Assembly

1. Remove the walker from its box and unwrap the parts.
2. Insert the push handles into the top of the front leg sections
3. Line up the holes on the push handles with the holes in the legs at the desired height and insert the bolts supplied.
4. Tighten the supplied hand knobs onto the bolts.

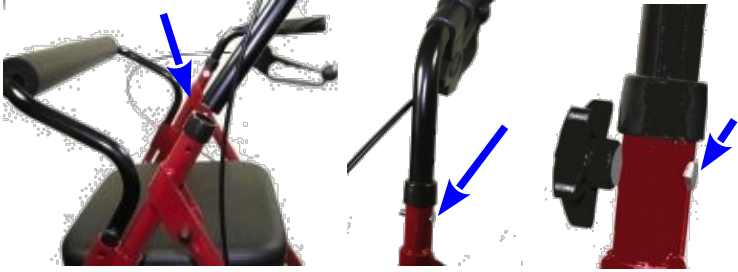

5. Insert the backrest into the backrest brackets on the front legs until they lock into place.



Instructions

Handle Height Adjustment

For proper height adjustment, stand upright with shoulders relaxed and arms at your side. Adjust handle height to the height of the inside of your wrist where your watch strap would be.



Folding

- 1) Lock the brakes on by pushing down both brake levers until they click into place.
- 2) Fold seat up vertically and lift strap attached to folding mechanism.
- 3) Optional: Backrest can be removed at any time by releasing the locking pins and lifting the backrest out of backrest bracket.



Unfolding

To unfold push down on the folding mechanism until they are fully extended.



Operating the Brakes

Always push down on both brake handles to lock the wheels before sitting. **CAUTION:** The rear wheels will lock but it may still slide if the floor surface is slippery.



Squeeze both brake handles to release the brake (2) **CAUTION** never release the brakes until you are standing stable and in the ready walk position. Do not release the brakes if you are sitting down.




If you are walking and wish to slow down, squeeze the brake handles slightly to put brake pressure on the wheels.



Brake Adjustment

Over time the brake cables will stretch and require adjustment.

- 1) With the brakes off, grasp the locking nut with a small wrench or pair of pliers, and turn counter clockwise to loosen.
- 2) With fingers, unscrew the adjusting screw to tighten brakes. A turn or two should be enough.
- 3) While holding adjustment screw in place tighten the locking nut.

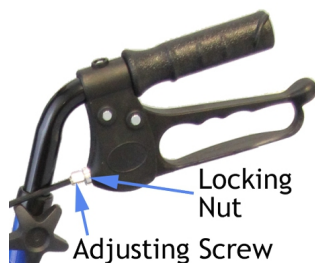

- 4) Repeat on brake on other side.

Note: If the brakes are too loose the wheels will turn when the brakes are on. If the cable is too tight the brakes will not lock on.

IMPORTANT: before using the walker, ensure the brakes are working properly. If the wheels turn while the brakes are applied have them adjusted before using the walker.

Safety Information

Consult a physician or other medical professional to determine if this equipment is suitable for your needs.

Ensure all parts are secure, brakes are functional and the frame is fully opened before using.

Always lock the brakes on before sitting and leave them locked until you're standing and ready to walk again.

Travel up and down slopes and inclines. DO NOT travel across inclines.

Use caution on slippery surfaces.

Do Not over tighten hardware attaching to the frame. This could cause damage to the frame tubing or strip the threads.

Do Not stand on seat.

Do Not overload this rollator. The weight capacity is 264 pounds (220 lb for Model CH3031) including the user and any things they may be carrying.

Inspection Items

Ensure rollator rolls straight (no excessive drag or pull to one side).

Inspect the seat for tears and/or excessive sagging.

Inspect hand grips for wear, looseness or damage.

Inspect for any loose or broken hardware or parts.

Inspect for cracked, broken or loose wheels.

Imported / Distributed by:

Factory Direct Medical

34 Futurity Gate, Unit # 15

Concord Ontario L4K 1S6

Local: 416-739-8393

Toll Free: 855-235-2400

